

# Power of **Fentanyl** Testing is in your hands!

First handheld FIA reader for Fentanyl testing approved by the FDA

510(k)  
cleared

Ultra Sensitive  
1 ng/mL



# BioSieve™ Toxismart™

# VivaChek

BioSieve™ / Toxismart™ / SalTox™




VivaChek Biotech (Hangzhou) Co., Ltd.  
Level 2, Block 2, 146 East Chaofeng Rd., Yuhang Economy  
Development Zone, Hangzhou, 311100, Zhejiang, P.R. China  
VivaChek Diagnostics, Inc.  
1401 21st St STE R Sacramento, California, 95811  
United States of America  
TEL: (888) 860-3894  
E-mail: info@vivachek.com  
Website: <https://vivachekdx.com>


Number: 1606017201  
Effective Date: 2024-06-28  
©2024 VivaChek Biotech (Hangzhou) Co., Ltd.

# 01 FIA TESTING SYSTEM


## Fentanyl Test Procedure

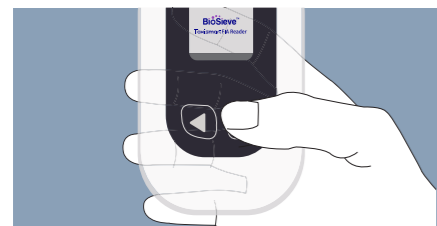


1 Press the right button "▶" for 5 seconds to turn on the Reader.

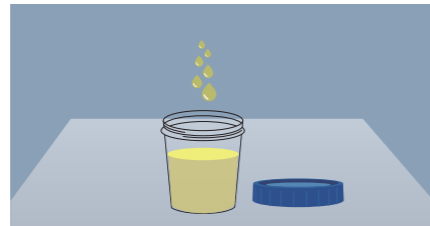

2 Collect the urine specimen.

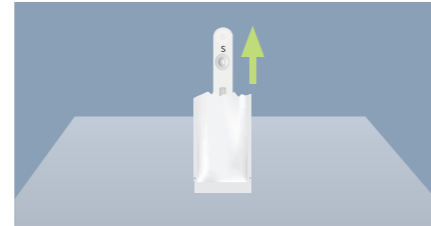

3 Take the Test Device out of the foil pouch and place it on a clean, dust-free and flat surface.

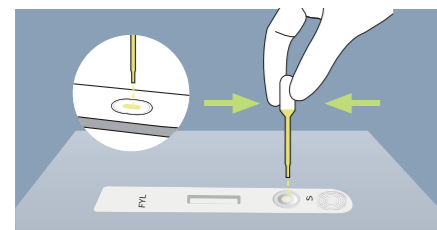

4 Transfer 3 drops of urine sample vertically to the sample well (S) using the dropper.



5 Insert the Test Device into the slot of the Reader immediately. Please ensure that orientation of the Test Device is as shown in the image above.



6 The Reader will automatically display the result after 5 minutes countdown.

## Product information

Product	BioSieve™ Fentanyl FIA Pro Test Kit
Specimen	Urine
Reaction time	5 min
Cut-off level	1.0 ng/mL

Product	BioSieve™ Toxismart FIA Reader
Size	124.5 × 72.5 × 40 mm
Weight	165 g
Connectivity	Bluetooth
Memory	999 results



## Ordering information

Item code (REF)	Description	Size
VCUS-FIARDR	BioSieve™ Toxismart FIA Reader	1Pcs/Box
VC-FYL1	BioSieve™ Fentanyl FIA Pro Test Kit	25T/Kit
VC-FYCTL	BioSieve™ Fentanyl Controls	1 Neg & 1 Pos/Box

# 02 OTHER LATERAL FLOW RAPID TEST

## Product information

Drug (Identifier)	Cut-off level (ng/mL)	Drug (Identifier)	Cut-off level (ng/mL)
Amphetamine (AMP)	1000 or 500	Morphine (MOP300/OPI2000)	2000 or 300
Secobarbital (BAR)	300	Methadone (MTD)	300
Buprenorphine (BUP)	10	Oxycodone (OXY)	100
Oxazepam (BZO)	300	Phencyclidine (PCP)	25
Cocaine (COC)	300 or 150	Propoxyphene (PPX)	300
2-ethylidene-1,5-dimethyl-3,3-diphenylpyrrolidine(EDDP)	300	Nortriptyline (TCA)	1000
Methylenedioxymethamphetamine(MDMA)	500	Marijuana (THC)	50
Methamphetamine (MET)	1000 or 500		

## Ordering information

Product	Number of Panel	Size
BioSieve™ Multi-Drug Urine Test Cup	Any combination of 2-15 items	25T/Box; 100Pcs/Carton
BioSieve™ Multi-Drug Urine Test Panel	Any combination of 1-15 items	25T/Box; 500Pcs/Carton
SalTox™ Multi-Drugs Saliva Rapid Test Cube	4-11 Drug Tests available for E&I; 2-14 Drug Tests available for Forensic Use Only	25T/Box; 100Pcs/Carton

